NOVOX

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CE 0477  
Datum revizije teksta:  
Rev. MOD-0922 od 13/09/2022.

NOVOX

## Medical device For the treatment of lesions in flat anatomical sites

**COMPOSITION:**  
Ingredients: oxygen-enriched olive oil (98%)  
Excipients: natural pear fragrance (2%)  
Support: polyurethane and polyester  
The device may contain traces (<0.4%) of the following allergens: CITRAL, GERANOL, CITRONELLO, LINALOOL, LIMONENE.

**PRODUCT DESCRIPTION**  
NovoX® Patch is a medical device for the treatment of surgical, traumatic or ulcerative lesions and scars in flat anatomical sites (such as abdomen, chest, back and thigh).  
NovoX® Patch, thanks to the creation of a micro-environment that is unfavourable for pathogen proliferation, and thanks also to the protective, barrier and soothing action of oxygen-enriched olive oil, NovoX® Patch aids the healing and correct cicatrization of wounds.  
NovoX® Patch is a medical device in the form of a single-use flat patch impregnated on one side with an oily gel.

**PACKAGE CONTENTS**  
1-piece box: 1 individually packaged impregnated substrate; 1 instruction manual (this leaflet)  
5-piece box: 5 individually packaged impregnated substrates; 1 instruction manual (this leaflet)  
10-piece box: 10 individually packaged impregnated substrates; 1 instruction manual (this leaflet)

**INTENDED USES**  
NovoX® Patch is intended for the treatment of:  
• Surgical wounds  
• Hypertrophic scars  
• Vascular and pressure ulcers  
• Ulcers due to neoplastic diseases  
• Post-radiotherapy ulcers and lesions  
• Ulcers or fistulas  
• Non-healing surgical wounds  
• Superficial, medium and deep burns  
• Traumatic wounds (e.g. abrasions, cuts and lacerations)  
NovoX® Patch is a non-sterile medical device. However, any previous contamination of the wound is not a contraindication to the use of the device, since tests have revealed the intrinsic capacity of the oxygen-rich oil to create an environment unfavourable to the proliferation of the pathogens commonly found on skin lesions, making it inherently protected against contamination by these pathogens.

**CONTRAINDICATIONS**  
NovoX® Patch must not be used on patients with known hypersensitivity or who have had an allergic reaction to the device or its components.  
NovoX® Patch must not be used in case of previous serious skin reactions to wound dressings.  
In case of known hypersensitivity to dressings, NovoX® Patch must only be used further to medical assessment.  
NovoX® Patch should not be used when complete coverage of the wound is not possible.

**PREGNANCY AND BREASTFEEDING**  
NovoX® Patch is a device for topical use not specifically tested for use during pregnancy. If you are or might be pregnant, we recommend that you do not use the device without medical advice.

**REMOVAL**  
• To remove the device, simply lift it off the lesion. If removal is difficult, saturate with saline solution or sterile water and lift slowly.

**DISPOSAL**  
NovoX® Patch must not be disposed of as unsorted waste. The device and any secondary dressings after use should be disposed of as potentially infected biological materials.

**STORAGE**  
Keep at a temperature below 25°C, in a cool, dry area far from sources of heat.  
In case of high room temperatures the product should be kept in the refrigerator (2-8°C).

CE 0477  
Date of last revision of product leaflet:  
Rev. MOD-0922 of 13/09/2022

	Keep at temperatures below 25°C	<b>REF</b>	Product number	<b>LOT</b>	Lot number		Expiry date - yyyy/mm/dd
<b>MD</b>	Medical device	<b>UDI</b>	Unique Device Identifier		Read the instructions for use prior to use		Do not reuse
	Read warnings prior to use		Do not use if the package is damaged		Keep away from light		Keep dry

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